ambIT+[®]

Pump Return Instructions

HOW TO RETURN YOUR AMBIT[®] PUMP

- 1 Turn off the pump by rotating the bottom counterclockwise.

If you have a shipping cap:

Remove the cassette. Then put on the shipping cap by lining up the tabs and snapping into place. Dispose of cassette, medibag and black carrying case.

2

If you did not receive a shipping cap:

Please leave cassette on and cut tubing with scissors. Dispose of medibag and black carrying case.

- 3 Slide the pump into the bubble-wrap sleeve provided and seal.

- 4 Place the pump and your completed patient questionnaire inside the shipping box and seal.

- 5 Make note of the tracking number on the front of the prepaid shipping box for your reference.

- 6 Ship the box via the service listed on the shipping label.

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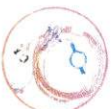
Date of Surgery:

Type of Surgery:

Pain level while on the pump: (please mark one for each day during your infusion 0=no pain, 10=worst pain)

Day 1:	0	1	2	3	4	5	6	7	8	9	10
Day 2:	0	1	2	3	4	5	6	7	8	9	10
Day 3:	0	1	2	3	4	5	6	7	8	9	10
Day 4:	0	1	2	3	4	5	6	7	8	9	10

Number of opioids taken each day:

Day 1:

Day 2:

Day 3:

Day 4:

Please rate your overall experience using the ambIT pump (1=terrible, 10=great)

Rating: 1 2 3 4 5 6 7 8 9 10

If you have any feedback that you would like to share about your experience with the pain pump, please feel free to leave a comment below. We use this survey data to help improve our product and service:

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ambIT+[®]

Frequently Asked Questions

1. Why is my pump making a grinding noise?

Your pump operates with a motor. The grinding noise is the pump delivering medication.

2. How much medication is the pump delivering?

Each time you hear the motor turn, the pump is delivering 50 microliters (0.05 mL) of medication.

3. How often will the motor on my pump run?

The pump will be dispensing a medicine at either a set rate or interval. Depending on what your doctor has prescribed, you may hear it turn a few times a minute or for a couple minutes every few hours and when the bolus is being delivered.

4. What is the bolus button?

The bolus button allows you to receive an extra dose of medication. The pump will infuse a set amount determined by your anesthesiologist. Push the round button to operate.

5. How often can I press the bolus button?

You can press the bolus button as often as you like. However, your pump will only deliver medication after a set period of time has elapsed from your last bolus delivery.

6. What does the green blinking light mean?

The green light indicates that your pump is running properly. If the pump was not working properly, the green light would no longer blink and the pump would make a loud alarming noise.

7. Who can I call if I have questions about my pump?

There is 24/7 support available for your pump, should you need it. The toll-free number is **800-444-2728**. Any questions regarding medication and dosing should be directed to your anesthesiologist. If you have a product complaint, please call

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